



Croc Surgical® Mat - Grippy Absorbent Matting

PRODUCT DETAILS

Never trip over a soggy hospital blanket again!
Switch to our Super Slim revolutionary grippy backed Croc Surgical®
Absorbent Mat.

Features / Benefits:

- **Advanced Grippy backing** — Equipped with a special grippy backing, the Croc mat stays firmly in place on dry floors during procedures, no matter what.
- **Highly absorbent** — our multi-layer pads absorb and retain 8-10 times its weight, 2.6 liters to 3.3 liters per m2 for 330g/m2 to help keep your O.R. floor clean and safe. *
- **Trip Free** — The mat lies flat on the floor, so there is no more tripping over bunched-up blankets and also allowing for easy movement of mayo stands and other carts.
- **Disposable** — save money with fast cost-effective cleanup and O.R. turnover while eliminating the need to launder heavy, liquid-soaked blankets.
- **Superior Durability & Composition** — mats won't rip, tear or fray under heavy foot traffic. Made with 330gm non-woven with 100% PP, adhesive film is 100% PE.
- **Perforated rolls** — easily create smaller pads by tearing at perforations.
- **Latex-free**
- **Self-extinguishing**
- **Mobile Mat Access** - Enhance accessibility by incorporating a wheeled, stainless steel mat roll dispenser for easy access.

*These numbers may change depending on the water quality, temperature and humidity at the test site, and test method.



Croc Surgical® Mat - Surgical Grippy Absorbent Matting

HOW TO USE:

Croc Surgical® Mat should only be applied to your floor before each procedure and left in place for the duration of the surgery.

1. Ensure the surface you want to cover is clean, dry and free of debris.
2. Position the mat on floor with the grippy backing side down.
3. Press down along the edges and into the center of the mat until it is smooth, flat, and securely in place, with no air pockets or creases.
4. After the procedure, simply lift and discard the mat. (Tip: Pinch an edge under the grippy bottom for effortless removal.)

WARNING: Used absorbents can take on the hazardous characteristics of liquids absorbed. Follow local, state and national regulations when storing or disposing of used absorbents.

NOTE: Croc Surgical® Absorbent Mats are designed for single use. Please dispose of properly.



Place with the grippy backing side down to eliminate bunching, curling or sliding during procedures.



Croc Surgical® Mat stays put to eliminate tripping or getting tangled in trolley or equipment wheels.